

Common Mouse Functions

Mouse Button	Function
Left	According to the operation mode.
Middle	Roll: scroll. Drag: windowing.
Right	Click: open context menu. Drag & Drop: swap viewports.
Left + middle	Pan.
Left + Right	Swivel.
Middle + Right	Zoom.
<Shift> Left mouse	Change thickness.
<Shift> Middle mouse	Change window width.
<Ctrl> Middle mouse	Change window center.
<Alt> Middle mouse	Change Opacity of Volume rendering.
<Ctrl> Left mouse	When using the mouse in the non-default mode (default is usually scroll or swivel) hold down control to activate the default mode.
<Ctrl> Pan + Left mouse	Vertically restricted pan (must be in pan mode).
<Alt> Pan + Left mouse	Horizontally restricted pan (must be in pan mode).
<Ctrl> + Left mouse	When selecting items with the left mouse, hold down <Ctrl> to Add/Remove items.
<Shift> + Left mouse	Select Range.

Common Keyboard Shortcuts

Function	Key	Function
Standard	<Ctrl>A	Select all in Directory.
	<Ctrl>C	Open Copy to dialog in Directory.
	Delete (Keyboard)	Delete selected items in Directory.
	F5	Refresh Directory.
Flip	<Ctrl>O	Rotate 180° vertical (flip orientation).
	O	Rotate 180° horizontal (flip orientation).
Orientation	A	Axial orientation.
	S	Sagittal orientation.
	C	Coronal orientation.
Annotation	D	Straight Line measurement
Film	F	Send image to film.
	<Shift>F	Send display to film.

Function	Key	Function
Report	R	Report images (sends images based on selection).
	<Shift>R	Report Display.
Save	<Shift>D	Save screen snapshot as ...
	<Shift>S	Save selected image(s) as ...
Viewing Tools	W	Windowing type-in dialog.
	T	Titles off/on.
	G	Grid off/on.
	I	Image parameters.
	M	Maximize/minimize the active viewport.
	B	Change background color for Volume Rendered image.
	X	Rotation Center Mode (slab) - the cursor will jump to the middle of view port.
	L	Link in Compare.
	<+>/<->	Increase/decrease zoom factor (zoom must be active).
	Page Up/Down	Scroll.
	Arrow Up/Down	Incremental scroll (functions like middle mouse button).
	Arrow Left/Right	Change thickness.
	Home/End	Scroll to beginning/end.
Windowing	0 through 9	Windowing presets (according to preferences).
Display Modes	<Shift> 1	Average.
	<Shift> 2	MIP.
	<Shift> 3	VIP.
	<Shift> 4	MinIP.
	<Shift> 5	Volume Rendering.
	<Shift> 6	Surface MIP.
Navigation	<Alt>N	Switch to Next series.

Function	Key	Function
Fusion Viewing	<Alt>Z	CCA - show hide overlay in Functional stage.
	P	In Dynamic Viewer function, switch between protocol presets available for currently loaded study. Press repeatedly if multiple presets exist.