

# MR Echo Accumulation



## Purpose

Use the Echo Accumulation package offered on the IntelliSpace Portal to cumulate multi echo sequences into one data set. This provides an image with exceptional Signal to Noise Ratio (SNR) and contrast to noise ratios.

This Quick Steps provides an optimal workflow scenario. Please keep in mind that this application has additional workflow steps and functionality, which are detailed in the Instructions for Use and is not intended to substitute or replace the Instructions for Use.



## Before you begin

Valid imaging series

The Echo Accumulation application requires the data set to have two or more echoes acquired.

# Workflow

To create a MR series with cumulated echoes:

1. Set the range of echoes for accumulation by dragging the ends of the Source range bar.
2. Click **Generate Series**
3. In the **File Name** field, type a name for the series.
4. Click **Generate Series**



© 2021 Koninklijke Philips N.V. All Rights Reserved. Reproduction or transmission in whole or in part, in any form or by any means, electronic, mechanical or otherwise, is prohibited without prior written consent of the copyright owner.

## Limitations of Liability

Philips has taken care to ensure the accuracy of these materials. However, Philips assumes no liability for errors or omissions and reserves the right to make changes without further notice to any products herein to improve reliability, function, or design. Philips may make improvements or changes in the product(s) or program(s) described in this content at any time.

## Educational use only

Philips is providing these materials for educational use only. The content is not intended to provide medical advice, diagnosis, or treatment. This content should not be regarded as a substitute or replacement for the Operating Manual or Instructions for Use. Before attempting to use any Philips medical devices, you must read the Instructions for Use thoroughly, paying particular attention to all warnings, cautions, and notes it may contain. In all cases the information contained in the Instructions for Use is considered leading information.

## Terms of use

Please see the full terms of use. [HTTP://philips.com/terms](http://philips.com/terms)

