



PHILIPS

Customer Services
Clinical Education

NM Viewer: Make Layouts







Purpose

This Quick Step provides an optimal workflow scenario. Please keep in mind this application has additional workflow steps and functionality, which are detailed in the *Instructions for Use*.

Workflow

To make layouts in the NM Viewer:

1. Select the study and series, and click  (**NM Viewer**).
2. In the **Tools** area, point to the  (drop-down arrow) and select **Layout**.
3. Click  (**Edit layout**) to access the layout creation tool.
4. In the bottom left corner, deselect the check box for **Keep Viewers**.
5. Under **Monitor 1** select **1x1** and under **Page** select how many viewers you want on Monitor 1.
 - If there is no second monitor, under **Monitor 2** select **0 x 0**.
6. Drag the page layout from **Layout editor** to the Monitor 1 image area.
7. If a dual monitor display is being created, repeat steps 5 and 6 for Monitor 2.
8. In the **Viewer type** field, select **Slab MPR** for TSC images or **Volume** for 3D rotating image.
9. Select **Overlay** images (PET) and **Underlay** images (CT).
10. Select **Fusion state** and **Orientation** to obtain images, then drag and drop the images into the appropriate viewer.
11. Repeat Steps 8 through 10 to populate the remaining viewers in the display area.
12. Click  (**Save layout as**). The **Save layout** dialog box displays.
13. In the **Name** box, type a unique name (entirely in lower case) for the layout with `.shared.pref` at the end to make the layout available to all IntelliSpace Portal users on your network (e.g. `mydisplay.shared.pref`).
14. For consistency, put the same name in the **Description** box.



Note

If `.shared.pref` is not placed at the end of the name, the layout will only be available to the user who created it. You must log out/in for the sharing to be active on the system. In addition, to add the new layout to the **Quick Layouts** in NM Viewer it must be added in the **PET Preferences**.

Educational Use Only

Philips is providing these materials for educational use only. They may not be sold or copied for distribution by any person, institution or company without the express written consent of Philips. Specifications are subject to change without notice.

Images shown in this guide are examples; the planning required should be discussed with the radiologist. **Warranty**

Disclaimer

Philips provides this document without warranty of any kind, implied or expressed, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Limitation of Liability

Philips has taken care to ensure the accuracy of this document. However, Philips assumes no liability for errors or omissions and reserves the right to make changes without further notice to any products herein to improve reliability, function, or design. Philips may make improvements or changes in the product(s) or program(s) described in this document at any time.



This material is not intended to substitute or replace the Operating Manual or *Instructions for Use*.

