



Saving and Naming a Secondary Capture




Purpose


By default, the IntelliSpace Portal NM Viewer does not have the Save as... option when saving secondary captures in NM Viewer. This Quick Step outlines how to select the name given to the saved secondary capture.

Please keep in mind that this application has additional workflow steps and functionality, which is detailed in the Instructions for Use and is not intended to substitute or replace the *Instructions for Use*.

Workflow

To Save and Name Secondary Captures:

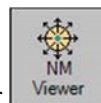

1. Load the appropriate data into the IntelliSpace Portal NM Viewer. The data can be dynamic, static, or whole body.
2. From the left navigation pane, click the  (drop-down arrow) and select **Batch**.
3. Scroll to starting image then click **Start** under **Define Batch**.
4. Scroll through images to end point and click **End** under **Define Batch**.
5. Click the **Save** drop-down arrow and select **Save Batch as....**
6. Name the batch.
7. Click the **File Format: DICOM** drop-down arrow and select one of the **Secondary Capture** options.
8. Select a **Destination**.
9. Click **Save**.

Educational Use Only

Philips is providing these materials for educational use only. They may not be sold or copied for distribution by any person, institution or company without the express written consent of Philips. Specifications are subject to change without notice.

Images shown in this guide are examples; the planning required should be discussed with the radiologist. **Warranty**

Disclaimer

Philips provides this document without warranty of any kind, implied or expressed, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose.

Limitation of Liability

Philips has taken care to ensure the accuracy of this document. However, Philips assumes no liability for errors or omissions and reserves the right to make changes without further notice to any products herein to improve reliability, function, or design. Philips may make improvements or changes in the product(s) or program(s) described in this document at any time.



This material is not intended to substitute or replace the Operating Manual or *Instructions for Use*.

